

Dragon Medical One Quick Reference Guide

Improving solution performance and user documentation efficiency

Using the DragonBar*



STANDBY MODE

Enter: Say *Go to sleep/Stop listening/Stop recording*
Exit: Say *Wake up*

Getting help

Say *Open help*
Say *What can I say*



Scan this QR code to access training videos, getting started materials and more!

* Your DragonBar may look different depending on your version of Dragon Medical One.

Microphone best practices

HEADSET

- Position the microphone just below your mouth, approximately 2" from chin
- Click the **red** microphone button on the DragonBar to talk
- Click the **green** microphone button on the DragonBar to stop recording or say **Stop Listening**
- Pause briefly before and after saying commands
- Speak in full sentences, including punctuation



POWERMIC

- Hold PowerMic in non-dominant hand
- Press and hold **red** button to dictate and keep it held while speaking
- Release the **red** button to turn microphone off

Correcting and deleting commands

- Scratch that
- Delete that
- Undo/Redo that
- Select <XYZ>
- Select that (selects last utterance)
- Select first/last/next word/all
- Select <word> through <word>
- Unselect that
- Correct <XYZ>
- Correct that
- Resume with <XYZ> (deletes text up to that word)

Inserting lines and spaces

- New paragraph
- New line
- Insert before/after <XYZ>

Capitalizing commands

- Cap that
- Cap <XYZ>
- All caps on/off
- All caps that

Navigating commands

- Go to top/bottom/end
- Start/end of field/sentence/paragraph
- Go back
- Insert before/after <word or phrase>

Dictating punctuation

Say this:	To type:
Comma	,
Period or full stop	.
Exclamation point/mark	!
Question mark	?
Colon	:
Semi colon	;
Open quote ... Close quote	" ... "
Open paren ... Close paren	(...)

Useful dictation box commands

- Open/close dictation box
- Transfer text
- Discard text



TRANSFER TEXT

Moves Dictation Box text to active field



RECALL TEXT

Brings back the last transferred text




DISCARD TEXT

Deletes contents of Dictation Box

Managing your vocabulary

ADD WORDS

1. Say *Add word* or say *Add that to vocabulary*
 2. Type correct spelling
- Note:** you can enter a new single word, acronym or a short phrase
3. Click Default Pronunciation or click the  mic icon to TRAIN

Note: It is not necessary to press the talk button to train a word

MANAGE YOUR CUSTOM WORDS

1. Say *Manage vocabulary* > search for the word/phrase to change/delete/train > correct as needed
- Note:** if the word/phrase has already been trained, delete and re-add to correct spelling
2. Click **+** to add a new word/phrase

Step-by-step commands

Step-by-step commands are used to execute commonly used actions in an application, or open applications

CREATE A STEP-BY-STEP COMMAND

1. Say *Manage Commands* or *Manage Step-by-step Commands*
2. Click **+** and enter a name
3. Enter a Description (optional)
4. Enter a Spoken Form
5. Click *New Step* and select the step type; repeat for each step/ keyboard shortcut.
6. Click *Apply All*

EDIT/DELETE A STEP-BY-STEP COMMAND

1. Say *Manage Commands* or *Manage Step-by-step Commands*
2. Select the Command you want to edit
3. Make desired changes (or click **X** to delete) and click *Apply All*

Auto-texts

Create auto-texts to bring in blocks of text with a simple voice command



ADD AN AUTO-TEXT:

1. Highlight desired text
2. Say *Make that an auto-text*
3. Type the name (what you will say to open it)
4. Click *Apply All* then click *Close*

USEFUL AUTO-TEXT COMMANDS:

- Next/Previous field
- First/Last field
- Accept defaults (clears all brackets [] and accepts ALL default text)
- Field complete (removes brackets [] and accepts text only in the CURRENT variable field)

Anchor dictation

1. Open Dragon Medical One and the target application (the program you are dictating into)
2. Place your cursor where you want the text to be typed
3. Say *Anchor Dictation*
4. The microphone icon changes to 
5. Begin dictating while you navigate through other applications
6. The microphone icon changes to 
7. Say *Release Dictation*



Exit and log off

- Access Log Off and Exit from the DragonBar Menu
- **Log Off** to change microphone sources or specialty vocabularies
- **Exit** when finished at a workstation to protect the profile from being accessed by other providers

Important reminders

- Remember to turn off the microphone when finished dictating
- Correct commonly misrecognized words to increase accuracy and save time
- Position the cursor where you want your text to go...your text will be typed in the window in which your cursor is blinking

Training and support resources

- To get help, say *What can I say*
- Click the Dragon flame  or hamburger icon  on the DragonBar* and select Help to explore the Dragon Help resources and discover information on Dragon Medical One functionality, tips, best practice advice and much more.
- Short instructional videos are available through Dragon Medical One training. Say *Open training* to access the course catalogue.
- Email CSO_Training_Help@microsoft.com to contact one of our Application Consultants.

* Your DragonBar may look different depending on your version of Dragon Medical One.

MEDICAL DEVICE DISCLAIMER. Nuance products are not designed, intended or made available as a medical device(s) and are not designed or intended to be a substitute for professional medical advice, diagnosis, treatment, or judgment and should not be used to replace or as a substitute for professional medical advice, diagnosis, treatment, or judgment.

Do Not Distribute. Contains Nuance Information shared under NDA.

© 2023 Microsoft. All rights reserved.
© 2023 Nuance Communications. All rights reserved.
HC_4604_PM4 NOV 2023